koogeek

BP2

QUICK START GUIDE

Upper Arm Blood Pressure Monitor

Safety Information

The Blood Pressure Monitor is ONLY for daily measurement, NOT for hypertension diagnosis.

NEVER conduct self-diagnosis or change the dosage of medication merely from the measurement result. Otherwise, may worsen your condition or cause death.

Please consult the doctor immediately when you get an abnormal result.

This device is intended for use in the adult population. Do not use this device on people who cannot express their intentions, including infants.

Caution

1. DO NOT use a cellular phone, or other devices that emit electromagnetic fields near this Monitor. This may result in incorrect measurements.
2. DO NOT forcibly collide, drop, stamper or shake the Monitor.
3. DO NOT disassemble, repair or modify the Monitor.
4. Please avoid alcoholic, droplets or other liquids getting into the inside of the Monitor.
5. Please discard the unusable battery in accordance with related environmental regulations.

1. Intended Use

The Monitor is designed to measure the systolic and diastolic blood pressure and pulse rate of adults.

2. How to Take Blood Pressure Measurement

2.1 Install Koogeek app and link it with the Monitor

2.1.1 Download and install the Koogeek app from <http://app.koogeek.com/download>.

2.1.2 Please register with an email account and log in.

2.2 Configuration - Link the Monitor with Koogeek app as the following instruction:

2.2.1 Launch the app.

- 2.2.2 Click "+" to add the device.
- 2.2.3 Choose the Blood Pressure Monitor List
- 2.2.4 Enter the configuration interface.
- 2.2.5 Click the configuration connection.
- 2.2.6 Link the Monitor with Koogeek app.

3. Start the Measurement

- 3.1 Turn on the Bluetooth.
- 3.2 Launch Koogeek app and enter the homepage.
- 3.3 Press the measurement button.
- 3.4 The cuff begins to inflate and the measurement starts.

Note:

1. In order to obtain correct measurement results, please sit tight, stay quite, and keep your hands, arms and fingers still when you are taking measurements.
2. Please put your elbow on the desktop, keep your palm facing up and make sure the wrist cuff at heart level.

4. After the measurement

- 4.1 The Monitor and Koogeek app will show the measurement results and save the data simultaneously.
- 4.2 If the user dose not launch the app, the measurements will be uploaded to the server via Wi-Fi and the Monitor will shut down automatically.

Note:

1. If the screen displays "E1", Pressure Speed is too slow or the gas circuit leaks too fast
2. If the screen displays "E2", Arm pressure over 300mmHg or measuring time over 180S
3. If the screen displays "E3", Can't detect the pulse signal during the measuring or measurement result is wrong.
4. If the heart rate indicator is always on the screen, it indicates the user's heart rate is abnormal.

5. Low battery indicator

5.1 The included rechargeable lithium batteries can support approximately 40 times of measurements after a full charge.


5.2 If the battery symbol displays only a third of battery, it suggests that the battery is in a low-battery state. Please charge it immediately.

6. Shut down

This Monitor will automatically shut down if no operation in 15 seconds.

7. Reset the Monitor

If the Monitor can not power on or take measurements, press the "Reset" button with an ejector to solve the problem.

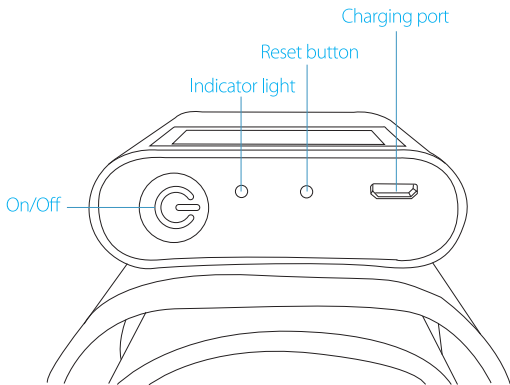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